



Product overview and intended use

The intended use of this product is for contraception and for the prevention of the transmission of sexually transmitted diseases. Remember, no method of contraception works 100% against pregnancy, HIV or sexually transmitted infections.

Performa™ Silicone lubricant contains 5% benzocaine, a local anaesthetic to help him last longer. Contains a medicinal substance.

Before you get going

Before you use this condom, check the expiry date on the condom wrapper. If the individual container is obviously damaged, throw that one away and use a new one from an undamaged package. Put a condom on before the penis touches the other person. This helps to prevent pregnancy and the possibility of catching sexually transmitted infections during vaginal, anal or oral sex. If pregnant or breastfeeding - ask a health professional before use.

How to use your condom

Tear the wrapper open from the serrated edge. Be careful with the condom, it could get damaged by fingernails or sharp objects.

With the roll outside, squeeze the teat and place the condom on the tip of the erect penis.

Roll down to the base. If the condom doesn't unroll, it may be on backwards, discard and use a new one. Stop and check if you feel the condom slipping off or too tight on the penis because this might lead to breakage. You should take the condom off soon after ejaculation. Hold the condom firmly at the base of the penis before pulling out.

Throw the foil and used condom into a bin. Don't flush down the toilet.

Contraindications

This condom is made of natural rubber latex and contains Benzocaine. If you have a known allergy to latex or Benzocaine, do not use this condom as this may cause allergic reactions including anaphylactic shock.

Warnings

STOP USE: If you feel discomfort or irritation while using condoms. Only use a condom once. Reuse increases risk of failure or infection. Condoms may present a choking hazard. Keep out of reach of children. If you use the condom for oral sex first, you should use a new condom for any other type of sex that follows.

Only use lubricants recommended for use with condoms. Oil-based ones (petroleum jelly, baby oil and some pessaries) and some topical medicines that go on the penis or vagina can damage condoms. For anal sex, use additional lubrication (do not use Durex Tingle or Embrace) on the outside of the condom. Speak to your doctor or pharmacist if you are unsure if you are allergic to latex or if irritation or discomfort persists after use or you are applying topical medicines to your penis or vagina. If a condom leaks or bursts during sex consult a pharmacist or doctor as soon as possible and within 72 hours. Premature ejaculation is a common problem, which can have many causes and may require medical consultation/supervision. If these condoms don't help, it's worth checking with your doctor.

For some people the Benzocaine Performa™ lubricant on these condoms can cause a rash, irritation, burning or itching. Don't get it in your eyes and wash it off after use. If you experience irritation, stop using Mutual Climax / Extended Pleasure (variant specific). If there's no improvement, see your doctor. Do not use these condoms if you or your partner has inflamed or broken skin. STOP USE IMMEDIATELY AND CALL A DOCTOR: If you or your partner experiences breathing difficulties or get blue lips when using these condoms.

Serious incidents must be reported to Reckitt and the local competent authority who will provide further advice.

Clinical performance

Helps to prevent pregnancy and the possibility of catching sexually transmitted infections during vaginal, anal or oral sex.

For Summary of Safety and Clinical performance, including additional information regarding pregnancy and STIs rates, see durex.co.uk/pages/summary-of-safety-and-clinical-performance. June 2025 update: The Summary of Safety and Clinical performance is available upon request to our customer care team

Condom failure: The clinical failure rate of latex condoms is approximately 2%; however, this statistic does not directly correlate with pregnancy rate.

When used correctly and consistently, condoms are highly effective in preventing pregnancy.

STI protection: Latex condoms reduce the risk of transmitting sexually transmitted infections (STIs) by providing a barrier against the source of infection.

Latex condoms are most effective against STIs that are spread via contact with the head of the penis, such as HIV.

Latex condoms are less effective against STIs which are spread by contact via infected skin not covered by a condom, such as Herpes.

If you believe you have an STI contact a health care provider.

The Summary of Safety and Clinical Performance is available upon request from our Customer Care team

UK - 0333 2005 345

RoI - 01 630 5429

Premature ejaculation is a common problem, which can have many causes and may require medical consultation/supervision. If these condoms don't help, it's worth checking with your doctor.

Legal manufacturer: Reckitt Benckiser Healthcare (UK) Limited, Dansom Lane, Hull, HU8 7DS, UK

Date of issue of the IFU: Rev 04 (06/2024)

RT-M-8Apqgk